
# Instructions for collecting Nasal Swab samples

Supervised by: Professor Kazuhiro Tateda,  
Department of Microbial and Infectious Diseases,  
Toho University School of Medicine


1

It is recommended for easier collection of the sample by pinching the area of swab 5-6 cm from the cotton ball.

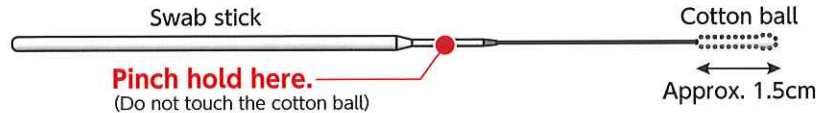
## 2 Insert the swab into one of nasal cavity.

- 1 Insert the swab into the nasal cavity **until the entire cotton ball is within the nasal cavity (Approx. 2 cm).**
- 2 Gently rub the mucous membrane of the nasal cavity by slowly **rotating the swab 5 times.**
- 3 **Wait 5 seconds** until the cotton ball is sufficiently moistened, then remove the swab carefully without touching other parts.
- 4 Using the same swab, repeat the same procedures for other nasal cavity.

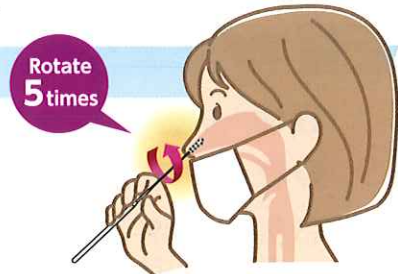Move the mask to reveal your nose.
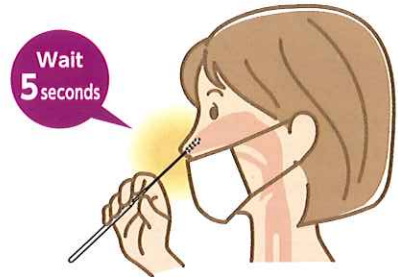
### Important

- Follow the instructions and guidance of doctors and medical staff.
- Do not touch the cotton ball. And please disinfect and wash your hands after the sample collection.
- Nasal cavities are prone to bleeding, so do not strongly rub or force the swab all the way in.
- Please be aware of droplets when sneezing.
- If you feel discomfort, discontinue collection and consult a doctor or medical staff immediately.

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